


2026

**BONE**<sup>®</sup>  
**SYSTEM**

paradigma<sup>®</sup>  
MONOPHASIC

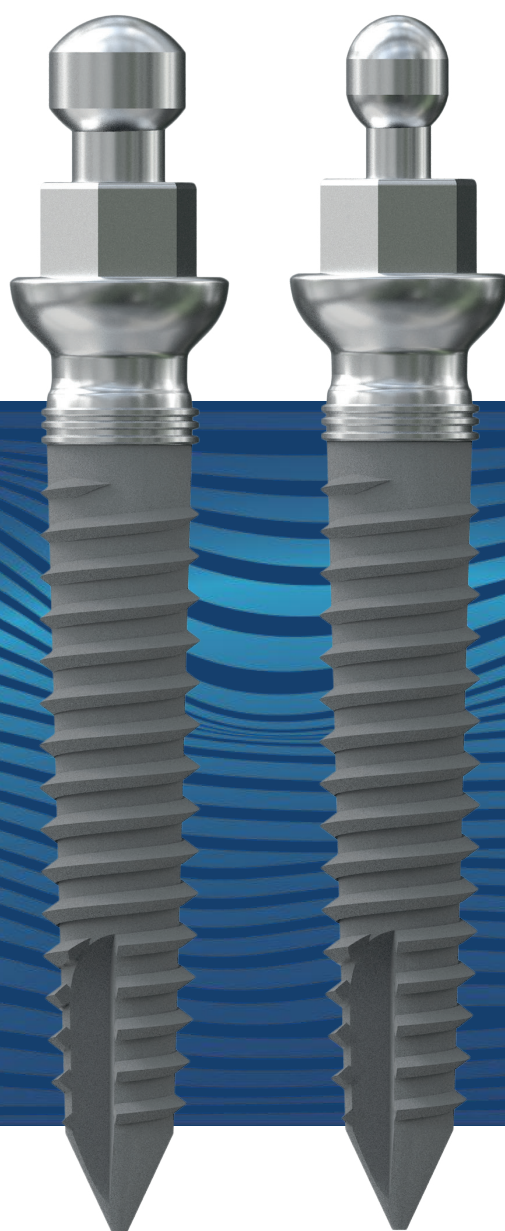


**CATALOG**



# Clinical Features

---

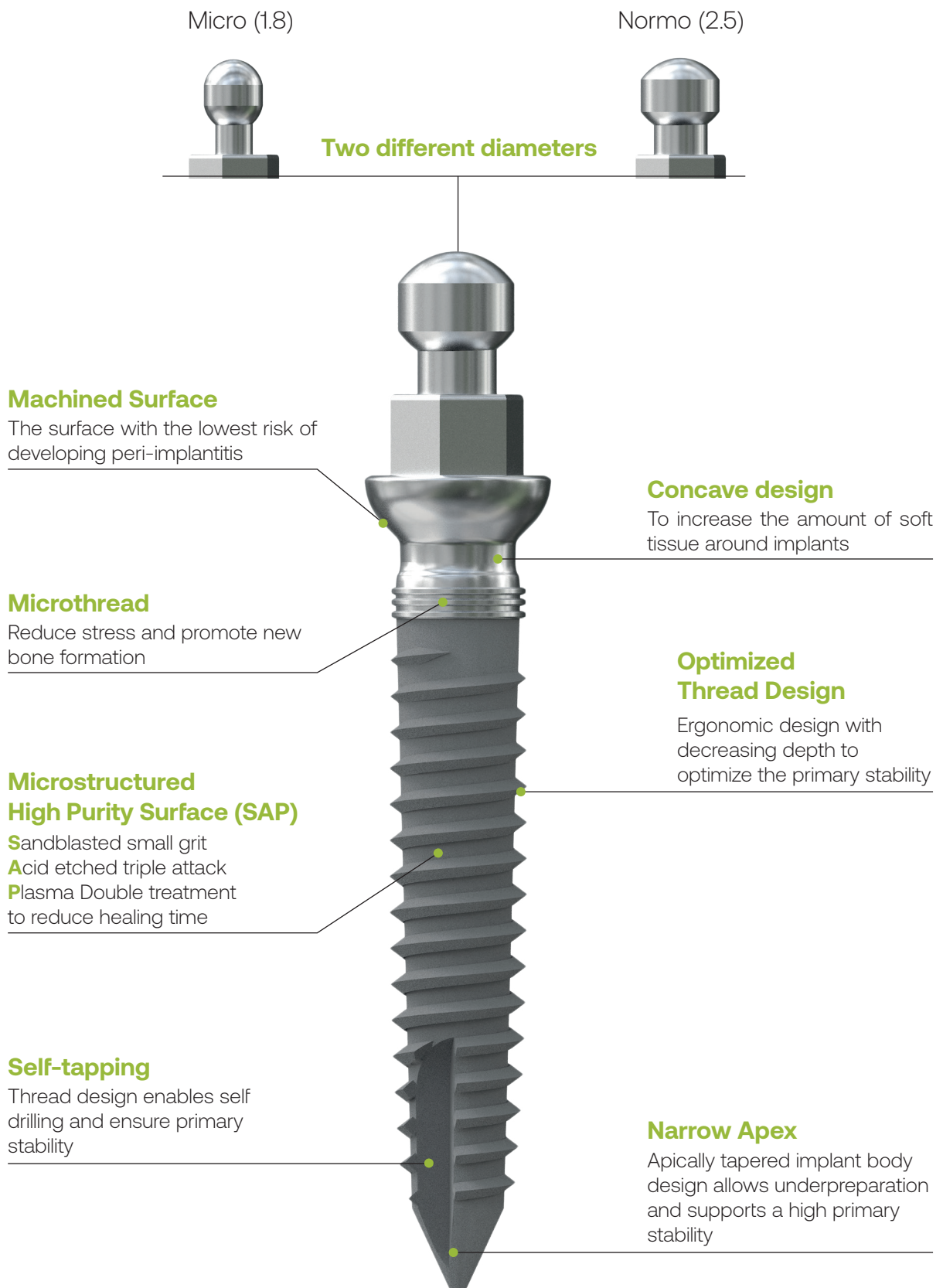


# Paradigma® Monophasic System

Paradigma® Monophasic Implant System offers one-piece implants with an O-Ball prosthetic connection. These are designed for narrow edentulous ridges and immediate treatment procedures in order to stabilize removable partial dentures or full removable overdentures.

The Paradigma® Monophasic Implant is made from Titanium Grade IV, treated with the SAP® surface treatment (Sandblasted, Acid Etched, Plasma Argon) and available in diameter Ø2.8 mm with length options of 11 mm, 13 mm, and O-Ball options of 1.8 mm (Micro) and 2.5 mm (Normo).

## Technical Specifications

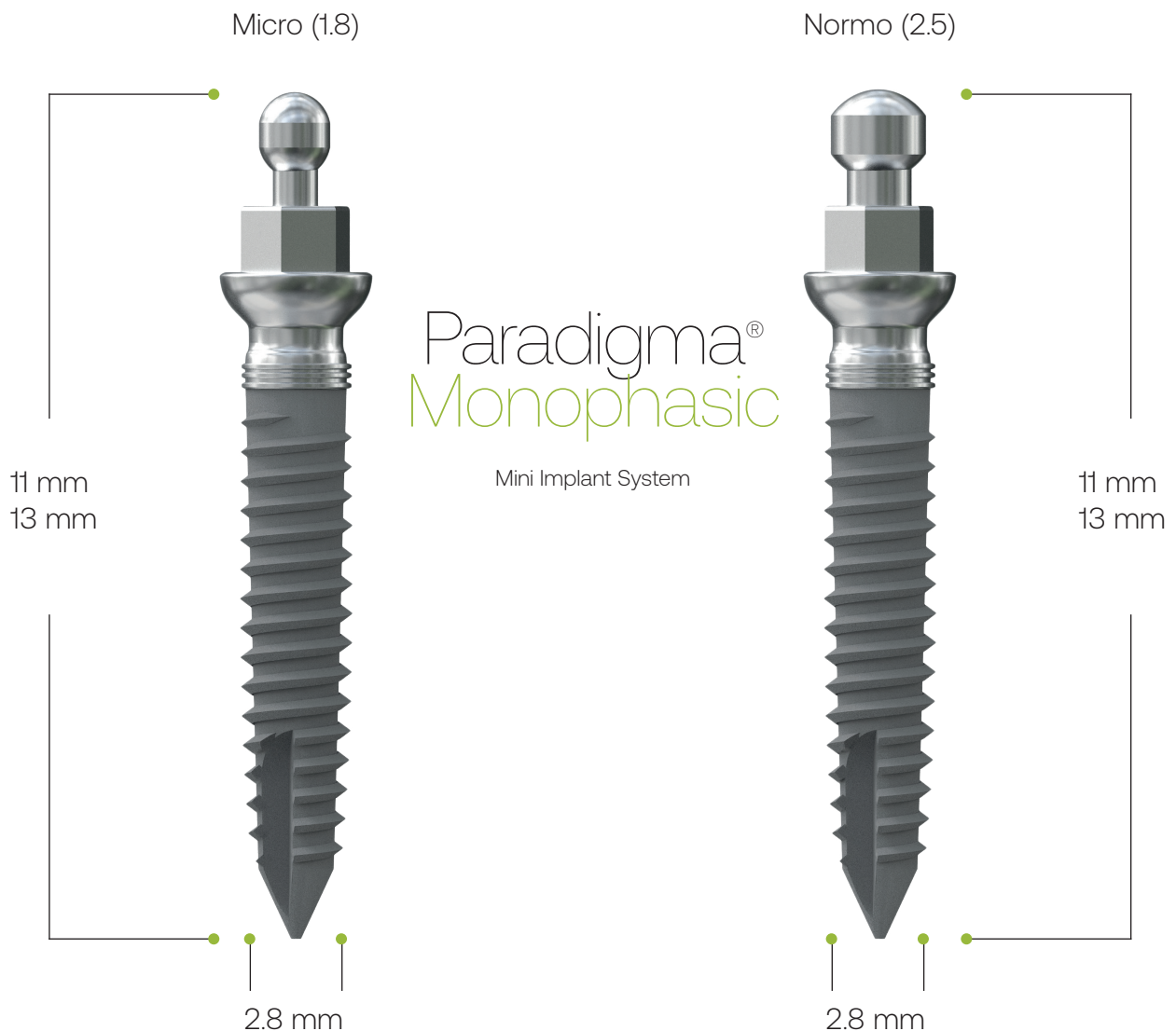

# Surgical Procedure

The workflow for the surgical procedure for the Paradigma® Monophasic Implant System involves 3 steps:


1. Preoperative planning
2. Implant bed preparation
3. Mini Implant placement

## 1. Preoperative Planning

After patient selection and evaluation protocols have been completed, the number of Paradigma® Monophasic Implants that should be placed are determined and thoroughly discussed with the patient. Information on bone availability for the implant bed of the patient and information of tissue depth mucosa thickness in the region of the prospected implant site by measuring with a perio probe should be available. After site selection, Paradigma® Monophasic Implants should be placed at least 5 mm apart and the space between them should be distributed equally.


## 2. Implant bed preparation

In order to obtain a good primary stability in most of clinical conditions it is advisable to respect the following drilling protocol.

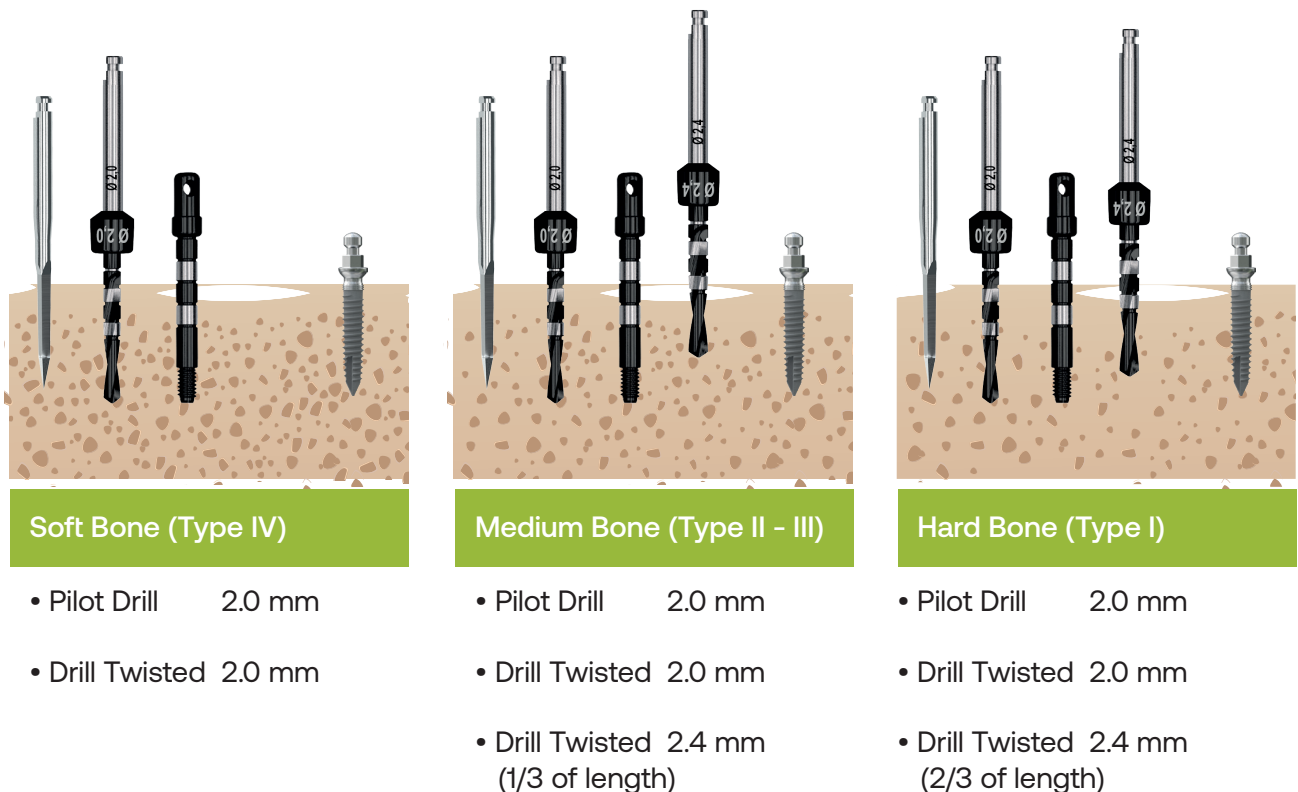


# Paradigma Monophasic Component

## Paradigma Monophasic Micro

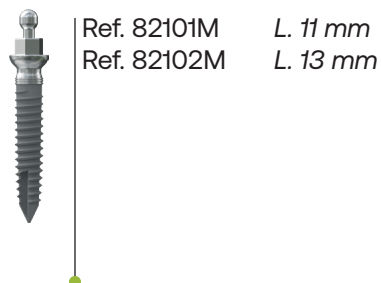

## Paradigma Monophasic Normo

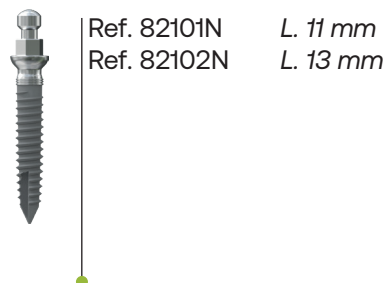


## Pilot Drill



## Initial Drills




## Parallel Pins

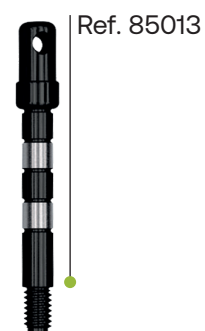



## Driver for contra angled



## Manual Driver

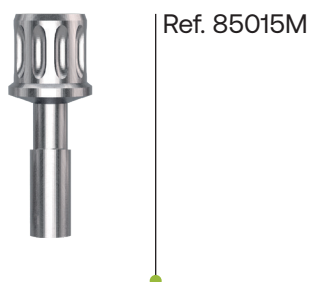

## Analogue

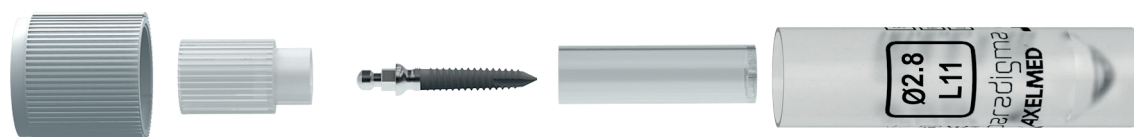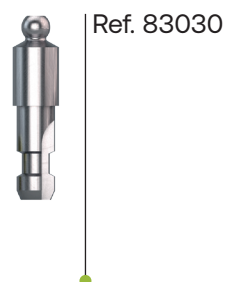



# PARADIGMA

[www.bonesystem.it](http://www.bonesystem.it)



 +39 02 2154165

 [bonesystem.it](http://bonesystem.it)

 [vendite@bonesystem.it](mailto:vendite@bonesystem.it)

BONE SYSTEM SRL

Via Rubattino 94/A

20134 Milano, Italia

Paradigma Mono EN  
Rev. 2026.09